

PLEASE READ ALL SECTIONS CAREFULLY

INSTALLATION:

Initially, the Steamer should be checked carefully for any transit damage. Please also check that the voltage shown on the rating plate (situated on the underside of the unit) matches that of your supply. If in doubt, consult a local electrical contractor, approved service engineer (see list enclosed) or REM Service Department direct (Tel. 01282 619977), before proceeding further.

Although a suitable three-pin plug is already fitted, it should be noted that, in the event of replacement, this must be fused at 5 amps only. (The maximum loading of the Steamer does not exceed 800 Watts). If, at any time, the mains supply lead is damaged, it must be replaced immediately. **DO NOT ATTEMPT TO USE UNTIL THIS WORK IS CARRIED OUT BY A SUITABLE CONTRACTOR, OR ENGINEER, AS PREVIOUSLY MENTIONED.** Inclusion of a 'Residual Current Device (RCD)', with a rated residual operating current not exceeding 30mA, in the mains circuitry of the Salon, will ensure extra safety and protection. (Again, 'expert' advice should be sought).

Warning: Do not attempt to gain access to the unit, or remove parts (other than those described in the Operating Instructions), without first disconnecting the unit from the mains supply. In any event, experienced personnel only, should carry out essential repairs. Pets and children must be kept well away from the Steamer whilst in use and any spillages cleared up with the unit fully disconnected.

The unit is supplied with a pole and base (plus reducer), which requires extreme care in both assembly and use. **DO NOT COMPRESS THE UPPER POLE UNTIL THE STEAMER IS IN POSITION.** (The weight of the unit will have the effect of providing a 'floating' situation, prior to setting same at the appropriate operating position). The required height can be secured via the butterfly nut.

However, the plastic beaker (Reservoir Bottle) can probably best be fitted with the Steamer Unit on its side prior to mounting on the pole and base. This can be accomplished by screwing the beaker into the opening on the underside and turning in a clockwise direction. To avoid steam or water escaping during operation, the fitting must be as tight as possible. This part of the installation is very important and should be observed each time the beaker is removed (e.g. for cleaning purposes).

WARNING: PLEASE NOTE THE MAXIMUM WATER LEVEL MARK ON THE RESERVOIR BOTTLE AND ALSO BEAR IN MIND THAT THIS ITEM WILL GET VERY HOT DURING OPERATION.



T: 01282 619977
F: 01282 619617

Facial Steamer: Installation and Operating Instructions
Page 1 of 3
Date: 30/09/2010 – Edition 1
Produced by: PP

OPERATING INSTRUCTIONS:

1. Ensure unit is switched off before filling with water. Remove filter cap on top of unit by turning either way and lifting at the same time. This is a very good fit and does appear to be very tight. (It will become easier with practice). An appropriate pouring vessel is vital in order to minimise spillage to the operating fascia and buttons. **ANY EXCESS WATER MUST BE WIPED AWAY BEFORE SWITCHING THE UNIT ON**. Don't forget the refit cap.

2. The 'on/off' operation of the Steamer is controlled via the 'Vapour' button. Watch for Steam appearing from the Dispenser Arm and utilise the 'Ozone' button, as required. A blue lamp (florescent) can be seen by viewing upwards from beneath the beaker when dispensing ozone. The steam nozzle can be turned through 360 degrees as usage demands.

N.B. Aromatherapy oils may be applied via the access flap at the top of the Dispenser Arm. This flap is hinged and may be opened by pushing it towards top of arm before lifting.

The following points are worthy of note in ensuring smooth and trouble-free use of this product:

- > Although there is an automatic cut-out device which activates if the water level drops below a certain point, constant reliance on this is not advised and should not be used as a means of controlling the water level. (The device itself and the heating element could suffer permanent damage if this advice goes unheeded). Always be aware of the water situation, by observation of the beaker contents. Please also allow a few minutes for the cut-out to re-set after filling with water, if the circumstances necessitate this.
- > Do not add Aromatherapy Oils via the water beaker.
- > Do not add pre-heated water (over 50 degrees C) to Reservoir Bottle.
- > Distilled Water may be required in 'hard water' regions.

CLEANING:

The body of the unit should be cleaned using non-spirit based and non-scouring type cleaner. Internal cleaning and/or servicing should be carried out by appropriate personnel. (see 'Installation' section at the head of these instructions).

ANY CLEANING CARRIED OUT DURING THE GUARANTEE PERIOD IS TOTALLY CHARGABLE AT THE DISCRETION OF THE SERVICE ENGINEER.

The logo for REM, consisting of the letters 'R', 'E', and 'M' in a large, bold, outlined font.

T: 01282 619977
F: 01282 619617

Facial Steamer: Installation and Operating Instructions
Page 2 of 3
Date: 30/09/2010 – Edition 1
Produced by: PP

DATA:

Serial No.: _____

Voltage: _____

GUARANTEE:

REM Facial Steamers are tested to the highest standards. However, should a fault develop within twelve months of **the date of purchase**, REM (UK) Ltd. undertake to effect any necessary repairs, including parts and labour, free of charge. The serial number of the unit **must** be quoted in all instances.

(Proof of purchase may be requested in certain circumstances)

The terms of this Guarantee do not affect your Statutory Rights.

IMPORTANT:

This apparatus is earthed and it is imperative that only a 3-pin plug be fitted and a 5-amp fuse used. The wires in this mains lead are coloured in accordance with the following code:


Green & Yellow - EARTH

Blue - NEUTRAL

Brown - LIVE

WARNING: THIS APPLIANCE MUST BE EARTHED

As the colours of the wires in the mains lead may not correspond with the coloured markings identifying the terminals in your plug, proceed as follows:

The wire which is coloured green-and-yellow must be connected to the terminal in the plug which is marked with the letter E or by the earth symbol  or coloured green or green-and-yellow.

The wire which is coloured blue must be connected to the terminal which is marked with the letter N or coloured black.

The wire which is coloured brown must be connected to the terminal which is marked with the letter L or coloured red.

REM

T: 01282 619977
F: 01282 619617

Facial Steamer: Installation and Operating Instructions

Page 3 of 3

Date: 30/09/2010 – Edition 1

Produced by: PP